**EN – ENGLISH - Instructions for use**

**Triangulation platform**  
Do not use Endomina if the package is damaged

**Terminology and package contents**

- A. Arm
- B. Active shoulder
- C. Passive shoulder
- D. Active leg
- E. Deployment leg
- F. Passive leg
- G. Foot
- H. Endoscope fixation wheel
- I. Arm pusher
- J. Brake
- K. Active therapeutic channel
- L. Passive therapeutic channel
- M. Lock connector

**Technical information**

Leg length – 860 mm  
Maximum diameter of the part inserted into the patient – 16,5 mm  
Shoulder length – 61 mm  
Maximum tool diameter – 1,9 mm  
Maximum duration of use – 4 hours

**Intended use**

Endomina is a short-term, single-use platform providing additional therapeutic channels for a flexible endoscope.

**Contraindications**

Women who are pregnant, trying to become pregnant or nursing; children; elderly; pre-existing conditions that may lead to fragile mucosa

*For oral insertion:*

- Any malformations from mouth to oesophagus (incl. pharynx)
- Any endoscopy contraindication

**Warnings**

In order to ensure the safe use of Endomina, it is essential to read this paragraph carefully.

- Before using Endomina, make sure that the package is properly sealed, make sure that the package has no hole or breach and includes all the components shown in the pictures and that they are in perfect condition. If any component is missing or damaged, do not use Endomina.
- Endomina is a device for gastro-enterology. It can only be used in the upper gastro-intestinal tract.
- Always keep at least one additional Endomina in stock before starting a procedure.
- Endomina can only be used by qualified medical staff (a physician or paramedical staff under surveillance by a physician) who has received the necessary training and certification to use it.
- This document describes the use of Endomina, not the clinical procedures that can be performed with it.
- Endomina is single-use and cannot be reused.
- Endomina can only be used on patients under general anaesthesia.
- Endomina accepts tools up to 1.9 mm. Do not use larger tools (see "Tool compatibility" for more details).
- Endomina can only be used with adult endoscopes (see "Endoscope compatibility" for more details).
- Endomina cannot be re-sterilised. Re-sterilising Endomina can provoke device failure. A re-sterilisation cycle may not be sufficient to completely eliminate small foreign bodies from hard-to-reach parts (e.g. in long and narrow channels). Moreover, after use, some mechanical components may not fulfil the initial specifications.

If Endomina is placed outside a clean zone, it cannot be used.

If one of the components of Endomina becomes detached and falls into a body cavity of the patient, retrieve it with endoscopic forceps.

Tools can only be inserted into the channels when the distal ends of the shoulders are visible with the endoscope and when the picture is clear and the arms are straight.

**Precautions**

- In all cases, an endoscopic examination is necessary prior to using Endomina in order to detect any malformation of the oesophagus.
- If perforated tissue is suspected, it is recommended that an endoscopy be performed to check the state of the tissue and take all necessary measures to treat any lesions prior using Endomina.
- Never insert Endomina in the open position.
- Never take Endomina out of the patient in the open position. Similarly, never take Endomina out of the patient without checking visually that Endomina is closed.

- EN – Instructions for use
- FR – Mode d'emploi
- NL – Gebruikshandleiding
- DE – Bedienungsanleitung
- IT – Istruzioni per l'uso

Never open/close Endomina without visual control.

Always keep the distal ends of the arms in view.

Never insert or remove Endomina while its arms are bent.

Always check that Endomina is not too large for the patient's natural orifice.

Never make sudden movements, especially during insertion and removal. This could hurt the patient.

Always wear appropriate protective clothing, as blood and other tissues carry an infection risk. Suitable protective clothing includes, but not limited to, protective goggles, a mask and waterproof clothes and gloves.

When using an electric tool (such as a diathermy knife), the other tools must have a nonconductive sheath to avoid electrical shocks for users.

Do not use force when inserting a tool. If there is resistance, straighten the arm until the tool can be inserted without effort. If this is still not the case, change the tool and/or check that its diameter is small enough for insertion into Endomina.

Do not kink Endomina's legs.

Do not use the endoscope in full retrovision when Endomina is fixed on it.

Always use CO<sub>2</sub> insufflation.

Endomina is damaged during the operation and a part stays in the stomach, take it out of the patient.

**Storage**

Endomina must be stored in its sterile package, in a clean and dry place, away from sun exposure and at ambient temperature. The sterile package should not be stored underneath other packages that could squash it.

**Repairs and modifications**  
Endomina has no parts that can be repaired by the user. Endomina must not be disassembled, modified or repaired by the user. This could hurt patients or damage other medical devices.

**Endoscope compatibility**  
Endomina can be used with any adult flexible endoscope with a diameter between 8.5 and 11.0 mm.

It has been validated with the endoscopes listed at the end of these instructions.

**Tool compatibility**  
Endomina can be used with TAPES. Devices with metal sheaths are easier to manipulate. It has been validated with the tools listed at the end of these instructions.

For other tools, please test prior to procedure. There is no guarantee that instruments selected solely using maximum insertion portion width and working length will be compatible in combination.

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**End of procedure**

1. Straighten the Endomina arm. If the arm remains bent, see the emergency procedure below.
2. Remove all tools from Endomina.
3. Open the Endomina shoulders.
4. Withdraw the endoscope by a few centimetres (2 to 5) to keep the end of Endomina in view.
5. Close the Endomina shoulders. If the shoulders remain open, see the emergency procedure below.
6. Gently remove the endoscope and Endomina from the patient.
7. Dispose of Endomina in an appropriate trash bin.

**Emergency procedures**  
If Endomina gets stuck in the open position inside the patient, use the following procedure:

1. Turn the endoscope fixation wheel as far as possible (in the opening direction, i.e. unscrewing)
2. Rotate the endoscope alternately + and – 90° until the endoscope is removed from Endomina.

3. If the shoulders can be closed now, continue the normal procedure. Otherwise, continue this procedure.

If Endomina is stuck in the open position, but not on the endoscope:

1. Withdraw Endomina by a few centimeters (2 to 5) to keep the end of Endomina in view.
2. Turn the endoscope fixation wheel as far as possible (in the closing direction, i.e. screwing in).
3. If the shoulders are closed enough to pass through the oesophagus (i.e. the space between the shoulders is about 3 mm or less), check that no parts are protruding and gently pull Endomina out, without using force.
4. If the shoulders are not closed enough to pass through the oesophagus (i.e. the space between the shoulders is still more than about 3 mm), slide the shoulders relative to each other by pushing the passive leg and pulling the handle. This will bend the muscles and bring the shoulders closer to each other. Check that no parts are protruding and gently pull Endomina out, keeping traction between the active and passive legs.

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**Storage**  
Endomina must be stored in its sterile package, in a clean and dry place, away from sun exposure and at ambient temperature. The sterile package should not be stored underneath other packages that could squash it.

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Product malfunction can be reported by e-mail to urgent@endotools.be. Please include your phone number in your e-mail.

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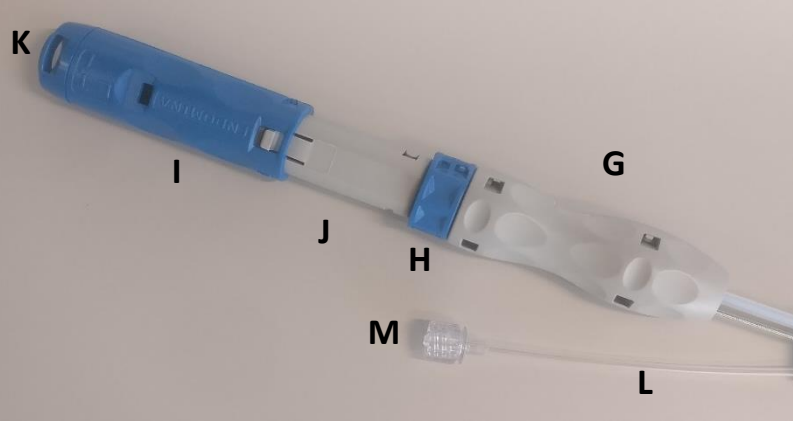
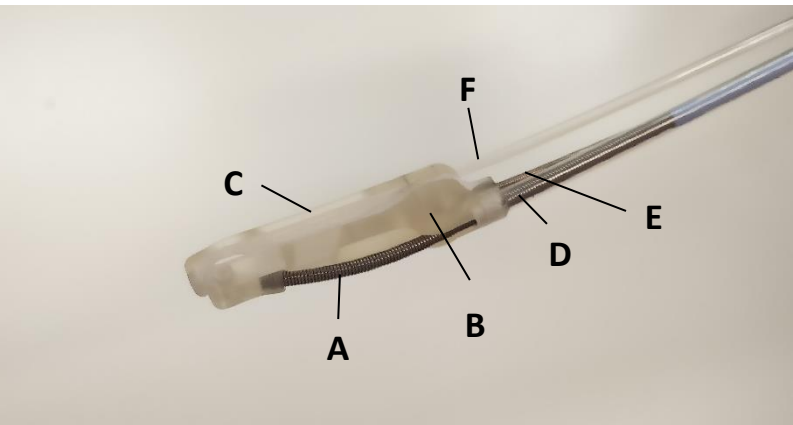
**EN – ENGLISH - Instructions for use**

**Triangulation platform**

Do not use Endomina if the package is damaged

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- Before using Endomina, make sure that the package is properly sealed, make sure that the package has no hole or breach and includes all the components shown in the pictures and that they are in perfect condition. If any component is missing or damaged, do not use Endomina.
- Endomina is a device for gastro-entology. It can only be used in the upper gastro-intestinal tract.
- Always keep at least one additional Endomina in stock before starting a procedure.
- Endomina can only be used by qualified medical staff (a physician or paramedical staff under surveillance by a physician) who has received the necessary training and certification to use it.
- This document describes the use of Endomina, not the clinical procedures that can be performed with it.
- Endomina is single-use and cannot be reused.
- Endomina can only be used on patients under general anaesthesia.
- Endomina accepts tools up to 1.9 mm. Do not use larger tools (see “Tool compatibility” for more details).
- Endomina can only be used with adult endoscopes (see “Endoscope compatibility” for more details).
- Endomina cannot be resterilized. Resterilising Endomina can provoke device failure. A reesterilisation cycle may not be sufficient to completely eliminate small foreign bodies from hard-to-reach parts (e.g. in long and narrow channels). Moreover, after use, some mechanical components may not fulfil the initial specifications.
- If Endomina is placed outside a clean zone, it cannot be used.
- If one of the components of Endomina becomes detached and falls into a body cavity of the patient, retrieve it with endoscopic forceps.
- Tools can only be inserted into the channels when the distal ends of the shoulders are visible with the endoscope and when the picture is clear and the arms are straight.

**Precautions**

- In all cases, an endoscopic examination is necessary prior to using Endomina in order to detect any malformation of the oesophagus.
- If perforated tissue is suspected, it is recommended that an endoscopy be performed to check the state of the tissue and take all necessary measures to treat any lesions prior using Endomina.
- Never insert Endomina in the open position.
- Never take Endomina out of the patient in the open position. Similarly, never take Endomina out of the patient without checking visually that Endomina is closed.
- Never open/close Endomina without visual control.
- Always keep the distal ends of the arms in view.

- EN – Instructions for use**
- ES – Modo de empleo**
- PT – Instruções de utilização**
- CS – Návod k použití**
- SR – Uputstvo za upotrebu**

2. Turn the endoscope fixation wheel as far as possible (in the closing direction, i.e. screwing in).
3. If the shoulders are closed enough to pass through the oesophagus (i.e. the space between the shoulders is about 3 mm or less), check that no parts are protruding and gently pull Endomina out, without using force.
4. If the shoulders are not closed enough to pass through the oesophagus (i.e. the space between the shoulders is still more than about 3 mm), slide the shoulders relative to each other by pushing the passive leg and pulling the handle. This will bend the muscles and bring the shoulders closer to each other. Check that no parts are protruding and gently pull Endomina out, keeping traction between the active and passive legs.

If Endomina is damaged during the operation and a part stays in the stomach, take it out of the patient.

**Storage**

Endomina must be stored in its sterile package, in a clean and dry place, away from sun exposure and at ambient temperature. The sterile package should not be stored squash it.

**Repairs and modifications**

Endomina has no parts that can be repaired by the user. Endomina must not be disassembled, modified or repaired by the user. This could hurt patients or damage other medical devices.

**Endoscope compatibility**

Endomina can be used with any adult flexible endoscope with a diameter between 9.2 and 10.8 mm.

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**Tool compatibility**

Endomina can be used with TAPES. Devices with metal sheaths are easier to manipulate. It has been validated with the tools listed at the end of these instructions.

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