**GB – ENGLISH - Instructions for use**

Do not use Endomina if package is damaged

**Terminology and package contents**

- A1. Arm
- A2. Active shoulder
- A3. Passive shoulder
- A4. Active leg
- A5. Deployment leg
- A6. Passive leg
- A7. Foot
- A8. Endoscope fixation wheel
- A9. Arm pusher
- A10. Brake
- A11. Active therapeutic channel
- A12. Passive therapeutic channel
- A13. Lock connector
- B. Mounting sheet

**Technical information**

- Legs length 860 mm
- Maximal diameter of the part 16 mm introduced in patient mouth
- Shoulders length 61 mm
- Maximal tool diameter – active 5 french leg and arm
- Maximal tool diameter – passive 5 french leg
- Maximal intervention duration 4 hours

**Intended use**

Endomina is a short-term, single-use catheter providing additional therapeutic channels to a flexible endoscope.

**Contra-indications**

- For oral introduction:
  - Esophageal malformations
  - Any endoscopy contra-indication
- For anal introduction:
  - Any endoscopy contra-indication

**Warnings**

In order to ensure a safe use of Endomina, it is essential to read this paragraph.

- Before using Endomina, make sure that package is properly sealed and includes all components represented on the pictures and that those are in perfect condition. If any component is missing or damaged, do not use Endomina.
- Endomina is a device for gastroenterology. It can only be used in natural orifices.
- Always keep an additional Endomina in stock before starting procedure.
- Endomina can only be used by qualified medical staff (a physician or paramedical staff under surveillance by a physician) who received necessary training and certification to use it.
- This document describes use of Endomina, not the clinical procedures that can be performed with it.
- Endomina is single use and cannot be reused.
- Endomina cannot be resterilized. A resterilization of Endomina can provoke device failure. A resterilization cycle may not be sufficient to completely eliminate small foreign bodies in the device (e.g. in long and narrow channels). Also, after use, some mechanical components may not fulfil initial specifications.
- Endomina can only be used on patients under general anesthesia.
- Endomina allows tools up to 5 french. Do not use larger tools (see tools compatibility for more details).
- Endomina can only be used with adult endoscopes (see endoscope compatibility for more details).
- If Endomina is placed out of clean zone, it cannot be used.
- If one component of Endomina falls in a patient cavity, retrieve it with an endoscopic forceps if it is judged dangerous for patient's health.
- Tools can only be introduced in the channels when the distal ends of the shoulders are visible with the endoscope and when the picture is clear and the arms are straight.

**Cautions**

- In any case, an endoscopic examination is necessary prior to using Endomina in order to detect any malformation of the introduction channel (esophagus for oral introduction).

- In case of doubt about a perforation, it is recommended to perform an endoscopy to check the state of the tissue and take all necessary measures to treat this lesion.
- Never introduce Endomina in open position.
- Never take Endomina out of the patient in open position. Similarly, never take Endomina out of the patient without checking visually that Endomina is closed.
- Never open / close Endomina without visual control.
- Always keep distal ends of the arms under visual control.
- Never introduce or take Endomina out whilst its arm is bent.
- Always check that Endomina is not too large for patient's esophagus.
- Never make sudden movements, especially during introduction and exit. This could hurt the patient.
- Always wear protective clothing, blood and other tissues carrying an infection risk. Suitable protective clothing includes protection glasses, mask and waterproof gloves and clothes.
- When using an electric tool (like a hot forceps), other tools must have a nonconductive sheath to avoid electrical shocks for users.
- Do not force when introducing a tool. If the tool is difficult to pass, bend then straighten the arm completely. If it is still hard to pass, change it and / or check that its diameter is compliant with Endomina.
- Do not bend Endomina legs.
- Do not use endoscope in full retrovision when Endomina is fixed on it.

**Adverse reactions**

The following adverse reactions have been observed during endoscopic interventions and may happen during Endomina use:

- Throat irritation

**Procedure**

Procedure described here is a check-list for an intervention done by an experienced user.

1. Check that Endomina package is not damaged and that sealing is perfect. If an anomaly is observed, do not use Endomina and send it back to Endo Tools Therapeutics.
2. For a procedure using oral introduction, perform an endoscopic exam to detect any malformation of the esophagus, which would block Endomina.
3. Unpack Endomina and check visually that it is complete and in good shape. Check that no part surface could cut the patient.
4. Check opening and closure of Endomina. Check arm bending.
5. Check that Endomina shoulders are closed. If needed, close them with the fixation wheel.
6. Check that the arm is straight. If needed, straighten it with the arm pusher.
7. Introduce two guidewires in the patient.
8. Introduce Endomina along the guidewires (in closed position) Use a lubrication gel if needed. Do not force.
9. Introduce the endoscope along Endomina channels.
10. Check that there is enough space to open Endomina.
11. Open Endomina with the endoscope fixation wheel.
12. Place endoscope between Endomina shoulders.
13. Close Endomina shoulders.
14. Perform the intervention.

**End of procedure**

1. Straighten Endomina arm. If it is not possible, see emergency procedure below.
2. Remove all tools from Endomina.
3. Open Endomina shoulders.
4. Move Endomina back of a few centimeters (2 to 5) to visualize Endomina shoulders.
5. Close Endomina shoulders. If it is not possible, see emergency procedure below.
6. Gently remove endoscope and Endomina from the patient.
7. Throw Endomina into an adequate trash bin.

**Emergency procedures**

If Endomina is stuck in open position inside the patient, use the following procedure: If Endomina is stuck on the endoscope:

1. Turn the endoscope fixation wheel to the maximum (towards opening, i.e. unscrew)
  2. Torque the endoscope alternatively + and – 90° until endoscope gets out of Endomina.
  3. If shoulders can be closed now, continue normal procedure. Otherwise, continue this one.
- If Endomina is stuck in open position, but not on the endoscope:
1. Move Endomina back of a few centimeters (2 to 5) to visualize Endomina shoulders.
  2. Turn the endoscope fixation wheel to the maximum (towards closing, i.e. screw).
  3. If shoulders are closed enough to pass through esophagus (i.e. space between shoulders is less than 3 mm), check that no part can hurt the patient and gently pull Endomina with visual control. Do not force.
  4. If shoulders are not enough closed to pass in the esophagus (i.e. space between shoulders is more than 3mm), slide the shoulders relatively to each other by pushing the passive leg and pulling the handle. This will bend the wires and bring shoulders closer to each other. Check that no part can hurt the patient and gently pull Endomina under visual control, keeping traction force between active and passive leg. Do not force.

If Endomina is damaged during intervention and a part stays in the stomach, take it out of the patient.

**Storage**

Endomina must be stored in its sterile package, in a clean and dry place, away from sun exposure and at ambient temperature. Sterile package cannot be crushed by other packages.

**Repair and modifications**

Endomina has no part that can be repaired by the user. Endomina cannot be disassembled, modified or repaired by the user. This could hurt patients or damage other medical devices.

**Endoscope compatibility**

Endomina can be used with any adult flexible endoscope with a diameter of 11 mm or less. It has been validated with endoscopes mentioned at the end of this IFU.

**Tools compatibility**

Endomina can be used with any endoscopic tool with a diameter of 5 French or less. It must be noted that metallic sheath devices are easier to manipulate. It has been validated with the tools mentioned at the end of this IFU.

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- Endo-Flex GmbH : fil guide pour bougie de dilatation 311 100