EN – ENGLISH - Instructions for use

Triangulation platform

Do not use Endomina if the package is damaged

Terminology and package contents

- A. Arm
- B. Active shoulder
- C. Passive shoulder
- D. Active leg
- E. Deployment leg
- F. Passive leg
- G. Foot
- H. Endoscope fixation wheel
- I. Arm pusher
- J. Brake
- K. Active therapeutic channel
- L. Passive therapeutic channel
- M. Lock connector

Technical information

Leg length – 860 mm
Maximum diameter of the part inserted into the patient – 16,5 mm
Shoulder length – 61 mm
Maximum tool diameter – 1,9 mm
Maximum duration of use – 4 hours

Intended use

Endomina is a short-term, single-use catheter providing additional therapeutic channels for a flexible endoscope.

Contraindications

women who are pregnant, trying to become pregnant or nursing; children; elderly;

For oral insertion:

- Oesophageal malformations
- Any endoscopy contraindication

For anal insertion:

- Any endoscopy contraindication

Warnings

In order to ensure the safe use of Endomina, it is essential to read this paragraph carefully.

- Before using Endomina, make sure that the package is properly sealed and includes all the components shown in the first picture and that they are in perfect condition. If any component is missing or damaged, do not use Endomina.
- Endomina is a device for gastroenterology. It can only be used in natural orifices.
- Always keep at least one additional Endomina in stock before starting a procedure.
- Endomina can only be used by qualified medical staff (a physician or paramedical staff under surveillance by a physician) who has received the necessary training and certification to use it.
- This document describes the use of Endomina, not the clinical procedures that can be performed with it.
- Endomina is single-use and cannot be reused.
- Endomina can only be used on patients under general anaesthesia.
- Endomina accepts tools up to 1.9 mm. Do not use larger tools (see "Tool compatibility" for more details).
- Endomina can only be used with adult endoscopes (see "Endoscope compatibility" for more details).
- Endomina cannot be resterilised. Resterilising Endomina can provoke device failure. A resterilisation cycle may not be sufficient to completely eliminate small foreign bodies from hard-to-reach parts (e.g. in long and narrow channels). Moreover, after use, some mechanical components may not fulfil the initial specifications.
- If Endomina is placed outside a clean zone, it cannot be used.
- If one of the components of Endomina becomes detached and falls into a body cavity of the patient, retrieve it with endoscopic forceps if it is judged dangerous for the patient's health.
- Tools can only be inserted into the channels when the distal ends of the shoulders are visible with the endoscope and when the picture is clear and the arms are straight.

Precautions

- In all cases, an endoscopic examination is necessary prior to using Endomina in order to detect any malformation of the insertion channel (oesophagus for oral insertion).
- If perforated tissue is suspected, it is recommended that an endoscopy be performed to check the state of the tissue and take all necessary measures to treat any lesions.
- Never insert Endomina in the open position.
- Never take Endomina out of the patient in the open position. Similarly, never take Endomina out of the patient without checking visually that Endomina is closed.

- Never open/close Endomina without checking visually.
- Always keep the distal ends of the arms in view.
- Never insert or remove Endomina while its arms are bent.
- Always check that Endomina is not too large for the patient's natural orifice.
- Never make sudden movements, especially during insertion and removal. This could hurt the patient.
- Always wear appropriate protective clothing, as blood and other tissues carry an infection risk. Suitable protective clothing includes protective goggles, a mask and waterproof clothes and gloves.
- When using an electric tool (such as a diathermy knife), the other tools must have a nonconductive sheath to avoid electrical shocks for users.
- Do not use force when inserting a tool. If there is resistance, straighten the arm until the tool can be inserted without effort. If this is still not the case, change the tool and/or check that its diameter is small enough for insertion into Endomina.
- Do not bend Endomina's legs.
- Do not use the endoscope in full retrovision when Endomina is fixed on it.

Adverse reactions

The following adverse reactions have been observed during endoscopic operations and may happen during the use of Endomina:

- Throat irritation

Procedure

The procedure described here is a check-list for an operation by an experienced user.

1. Check that the Endomina package is not damaged and is perfectly sealed. If any anomaly is detected, do not use Endomina; send it back to Endo Tools Therapeutics.
2. For a procedure using oral insertion, perform a prior endoscopic exam to detect any malformation of the oesophagus, which would block Endomina.
3. Unpack Endomina and check visually that it is complete and in good condition. Check that no parts protrude.
4. Check that Endomina opens and closes. Check the action of the arm.
5. Check that Endomina's shoulders are closed. If not, close them with the endoscope fixing wheel.
6. Check that the arm is straight. If not, straighten it with the arm pusher.
7. Insert two guide wires into the patient.
8. Insert Endomina gently into the patient along the guide wires (in closed position). Use a lubricating gel if needed. Do not use force.
9. Insert the endoscope along Endomina's channels.
10. Check visually that there is enough space to open Endomina.
11. Open Endomina with the endoscope fixing wheel.
12. Place the endoscope between the Endomina shoulders.
13. Close the Endomina shoulders.
14. Perform the operation.

End of procedure

1. Straighten the Endomina arm. If the arm remains open, see the emergency procedure below.
2. Remove all tools from Endomina.
3. Open the Endomina shoulders.
4. Withdraw the endoscope by a few centimetres (2 to 5) to keep the end of Endomina in view.
5. Close the Endomina shoulders. If the shoulders remain open, see the emergency procedure below.
6. Gently remove the endoscope and Endomina from the patient.
7. Dispose of Endomina in an appropriate trash bin.

Emergency procedures

If Endomina gets stuck in the open position inside the patient, use the following procedure:

- If Endomina is stuck on the endoscope:
1. Turn the endoscope fixing wheel as far as possible (in the opening direction, i.e. unscrewing)
 2. Rotate the endoscope alternately + and – 90° until the endoscope is removed from Endomina.
 3. If the shoulders can be closed now, continue the normal procedure. Otherwise, continue this procedure.
- If Endomina is stuck in the open position, but not on the endoscope:
1. Withdraw Endomina by a few centimetres (2 to 5) to keep the end of Endomina in view.
 2. Turn the endoscope fixing wheel as far as possible (in the closing direction, i.e. screwing in).

3. If the shoulders are closed enough to pass through the oesophagus (i.e. the space between the shoulders is about 3 mm or less), check that no parts are protruding and gently pull Endomina out, without using force.
4. If the shoulders are not closed enough to pass through the oesophagus (i.e. the space between the shoulders is still more than about 3 mm), slide the shoulders relative to each other by pushing the passive leg and pulling the handle. This will bend the muscles and bring the shoulders closer to each other. Check that no parts are protruding and gently pull Endomina out, keeping traction between the active and passive legs.

If Endomina is damaged during the operation and a part stays in the stomach, take it out of the patient.

Storage

Endomina must be stored in its sterile package, in a clean and dry place, away from sun exposure and at ambient temperature. The sterile package should not be stored underneath other packages that could squash it.

Repairs and modifications

Endomina has no parts that can be repaired by the user. Endomina must not be disassembled, modified or repaired by the user. This could hurt patients or damage other medical devices.

Endoscope compatibility

Endomina can be used with any adult flexible endoscope with a diameter of 11 mm or less. It has been validated with the endoscopes listed at the end of these instructions.

Tool compatibility

Endomina can be used with any endoscopic tool with a diameter of 1.9 mm or less. Devices with metal sheaths are easier to manipulate. It has been validated with the tools listed at the end of these instructions. For other tools, please test prior to procedure. There is no guarantee that instruments selected solely using maximum insertion portion width and working length will be compatible in combination.

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