**EN – ENGLISH - Instructions for use**

**Triangulation platform**

Do not use Endomina if the package is damaged

**Terminology and package contents**

- A. Arm
- B. Active shoulder
- C. Passive shoulder
- D. Active leg
- E. Deployment leg
- F. Passive leg
- G. Foot
- H. Endoscope fixation wheel
- I. Arm pusher
- J. Brake
- K. Active therapeutic channel
- L. Passive therapeutic channel
- M. Lock connector

**Technical information**

Leg length – 860 mm  
Maximum diameter of the part inserted into the patient – 16,5 mm  
Shoulder length – 61 mm  
Maximum tool diameter – 1,9 mm  
Maximum duration of use – 4 hours

**Intended use**

Endomina is a short-term, single-use catheter providing additional therapeutic channels for a flexible endoscope.

**Contraindications**

women who are pregnant, trying to become pregnant or nursing; children; elderly; pre-existing conditions that may lead to fragile mucosa

**For oral insertion:**

- Oesophageal malformations
- Any endoscopy contraindication

**For anal insertion:**

- Any endoscopy contraindication

**Warnings**

In order to ensure the safe use of Endomina, it is essential to read this paragraph carefully.

- Before using Endomina, make sure that the package is properly sealed and includes all the components shown in the first picture and that they are in perfect condition. If any component is missing or damaged, do not use Endomina.

- Endomina is a device for gastro-entereology. It can only be used in natural orifices.

- Always keep at least one additional Endomina in stock before starting a procedure.

- Endomina can only be used by qualified medical staff (a physician or paramedical staff under surveillance by a physician) who has received the necessary training and certification to use it.

- This document describes the use of Endomina, not the clinical procedures that can be performed with it.

- Endomina is single-use and cannot be reused.

- Endomina can only be used on patients under general anaesthesia.

- Endomina accepts tools up to 1.9 mm. Do not use larger tools (see "Tool compatibility" for more details).

- Endomina can only be used with adult endoscopes (see "Endoscope compatibility" for more details).

- Endomina cannot be resterilised. Resterilising Endomina can provoke device failure. A resterilisation cycle may not be sufficient to completely eliminate small foreign bodies from hard-to-reach parts (e.g. in long and narrow channels). Moreover, after use, some mechanical components may not fulfil the initial specifications.

- If Endomina is placed outside a clean zone, it cannot be used.

- If one of the components of Endomina becomes detached and falls into a body cavity of the patient, retrieve it with endoscopic forceps if it is judged dangerous for the patient's health.

- Tools can only be inserted into the channels when the distal ends of the shoulders are visible with the endoscope and when the picture is clear and the arms are straight.

**Precautions**

- In all cases, an endoscopic examination is necessary prior to using Endomina in order to detect any malformation of the insertion channel (oesophagus for oral insertion).

- If perforated tissue is suspected, it is recommended that an endoscopy be performed to check the state of the tissue and take all necessary measures to treat any lesions.

- Never insert Endomina in the open position.

- Never take Endomina out of the patient in the open position. Similarly, never take

EN – Instructions for use

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2. Turn the endoscope fixing wheel as far as possible (in the closing direction, i.e. screwing in).
3. If the shoulders are closed enough to pass through the oesophagus (i.e. the space between the shoulders is about 3 mm or less), check that no parts are protruding and gently pull Endomina out, without using force.
4. If the shoulders are not closed enough to pass through the oesophagus (i.e. the space between the shoulders is still more than about 3 mm), slide the shoulders relative to each other by pushing the passive leg and pulling the handle. This will bend the muscles and bring the shoulders closer to each other. Check that no parts are protruding and gently pull Endomina out, keeping traction between the active and passive legs.

If Endomina is damaged during the operation and a part stays in the stomach, take it out of the patient.

**Storage**

Endomina must be stored in its sterile package, in a clean and dry place, away from sun exposure and at ambient temperature. The sterile package should not be stored underneath other packages that could squash it.

**Repairs and modifications**

Endomina has no parts that can be repaired by the user. Endomina must not be disassembled, modified or repaired by the user. This could hurt patients or damage other medical devices.

**Endoscope compatibility**

Endomina can be used with any adult flexible endoscope with a diameter of 11 mm or less. It has been validated with the endoscopes listed at the end of these instructions.

**Tool compatibility**

Endomina can be used with any endoscopic tool with a diameter of 1.9 mm or less. Devices with metal sheaths are easier to manipulate. It has been validated with the tools listed at the end of these instructions.

For other tools, please test prior to procedure. There is no guarantee that instruments selected solely using maximum insertion portion width and working length will be compatible in combination.

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