EN – ENGLISH – Instruction for use

Do not use if package is damaged

Terminology and package contents

- A. Fixation handle
- B. Needle pusher
- C. Tag pushers handle
- D. Distal tag pusher
- E. Proximal tag pusher
- F. Distal safety pin
- G. Proximal safety pin
- H. Brake
- I. Needle and tags (in the dispenser)
- J. Dispenser attachment

Technical information

Needle diameter	1.75 mm / 5.25Fr
Needle rigid length	17 mm
Total needle length (excl. Handle)	1200 mm
Tags diameter	1.1mm / 3.3Fr
Tags lengths	10 mm
Suture	EP1.5 (USP 4-0) braided polyester
Maximum intervention duration	4 hours

Intended use

TAPES is a long-term, single use suturing device intended to pierce and link two tissues together using a flexible endoscope and a triangulation platform.

Contra-indications

Any contra-indication to endoscopy ; gastric ulcer ; perforations ; severe gastritis ; bleeding ; esophagitis ; colitis ; hiatal hernia ; pregnant women or willing to become pregnant ; addictions (drug or alcohol) ; rectitis ; varices ; stomach portal hypertension ; fibrotic sites

Warnings

In order to ensure a safe use of TAPES, it is essential to read this paragraph.

- Before using TAPES, make sure that package is properly sealed and includes all components represented on the pictures and that those are in perfect condition. If any component is missing or damaged, do not use TAPES.
- TAPES is a device for gastro-entérology. It can only be used in natural orifices.
- Always keep an additional TAPES in stock before starting procedure.
- TAPES can only be used by qualified medical staff (a physician or paramedical staff under surveillance by a physician) who received necessary training and certification to use it.
- This document describes use of TAPES, not the clinical procedures that can be performed with it.
- TAPES is single use and cannot be reused, resterilized or reloaded.
- TAPES can only be used on patients under general anesthesia.
- TAPES can only be used with triangulation platforms (i.e. not directly in the endoscope). See "triangulation platform compatibility" for more detail.
- Triangulation platform used must be able to form big enough plicatures for the suture to hold.
- If TAPES is placed out of clean zone, it cannot be used.
- If one of the components of Endomina falls in a patient cavity, retrieve it with an endoscopic forceps if it is judged dangerous for patient's health.

Cautions

- Never make sudden movements, especially during introduction and exit. This could hurt the patient.

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- The needle cannot be apparent during introduction in the triangulation platform. This could hurt the patient.
- Tapes must always be manipulated under visual control when it is out of the triangulation platform, in the cavity to treat.
- Always wear protective clothing, blood and other tissues carrying an infection risk. Suitable protective clothing includes gloves, protection glasses, mask and waterproof clothes and gloves.
- Do not force when introducing TAPES. If it is difficult to pass, remove it and investigate.
- When multiple tag chains are used, avoid that suture cross each other.
- Do not insert TAPES in a bent channel. Always introduce TAPES in a channel as straight as possible.
- TAPES is a 5.25 Fr device. Use it only in a suitable channel (up to 7 Fr). To use it in a larger channel, use a catheter to avoid guiding issues. TAPES should be tested before usage in a 5 Fr channel.
- Always keep handle firmly in hand to avoid undesired movements.

Adverse reactions

The following adverse reactions have been observed during endoscopic interventions involving needles and may happen after TAPES use:

- Nausea, vomiting, cramps
- Bleeding
- Minor peritonitis
- Minor bacteremia

Procedure

Procedure described here is a check-list for an intervention done by an experienced user.

1. Check that TAPES package is not damaged and that sealing is perfect. If an anomaly is observed, do not use TAPES and send it back to Endo Tools Therapeutics.
2. Introduce a compatible triangulation platform to the site to treat.
3. Unpack TAPES and check visually that it is complete and in good shape. Check that tags are in place and that handle has not been actuated. Remove TAPES from dispenser (torque the handle)
4. Check sliding of needle pusher on fixation handle and check the brake screw.
5. Gently introduce TAPES in the triangulation platform using lubricating gel if necessary. Do not force. Check that platform angulation allows introduction.
6. If necessary, assemble TAPES to triangulation platform with the connector.
7. Actuate triangulation platform to position TAPES needle correctly.
8. Check visually that TAPES needle slides correctly out of the platform.
9. Set brake at the maximum admissible piercing position.
10. Pierce first tissue by pushing handle.
11. Remove distal safety pin and drop distal tag using distal tag pusher.
12. Pull back distal tag pusher flush with the proximal tag pusher.
13. Pull needle back in the platform by pulling handle.
14. Move platform to loose suture if necessary.
15. Pierce second tissue.
16. Remove proximal safety pin and drop proximal tag using proximal tag pusher. Check that all suture has been released. Make back and forth movements with the pusher if necessary.
17. Pull back proximal tag pusher.
18. Pull back needle in the platform.
19. Straighten triangulation platform to allow needle removal. Always keep visual control.
20. Take TAPES out of platform. Do not force.

21. With a rat tooth forceps in a catheter or similar combination, tighten tag chain by pulling on suture loop. Tighten until tags exert suitable pressure on the tissues.
22. With a cutting tool, cut free end of the suture. Do not cut suture between tags.

Procedure end

1. Check that all components are present (excluding tags and suture).
2. Dispose TAPES in an adequate trash bin (for sharp elements).

Emergency procedure

If TAPES is stuck in triangulation platform or tissue, use the following procedure:

If TAPES is stuck in tissue:

1. Drop proximal tag to release needle from tissues.
2. If necessary, cut tag or suture with a stamp cutter or endoscopic scissor to release needle.
3. If necessary, retrieve tags and suture with an endoscopic forceps.

If TAPES is stuck in triangulation platform:

1. Pull completely the distal and proximal tag handles.
2. Check that platform angulation allows TAPES sliding.
3. If TAPES is still stuck, if needle does not get out and triangulation platform allows, retrieve everything altogether (platform and TAPES) and investigate reasons for blocking.
4. If needle is stuck out of channel or platform does not allow combined retrieval, cut the needle e.g. with argon plasma.
5. Retrieve the needle with a foreign body forceps.
6. Remove TAPES from platform.

If TAPES is damaged during intervention and a part stays in the stomach, take it out of the patient.

Storage

TAPES must be stored in its sterile package, in a clean and dry place, away from sun exposure and at ambient temperature. Sterile package cannot be crushed by other packages.

Repair and modifications

TAPES has no part that can be repaired by the user. Endomina cannot be disassembled, modified or repaired by the user. This could hurt patients or damage other medical devices.

Triangulation platform compatibility

TAPES can be used with any triangulation platform with a large enough channel (5.25 Fr or more). It has been validated with Endomina V2 from Endo Tools Therapeutics.

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