EN – ENGLISH – Instructions for use

Endoscopic tissue approximation device

Do not use if package is damaged

Terminology and package contents

- Fixing handle
- Needle pusher
- Distal tag pusher
- Proximal tag pusher
- Safety pin
- Brake
- Needle and tags (in the dispenser)
- Dispenser attachment

Technical information

Maximum diameter – 1,9 mm
Needle diameter – 1,75 mm
Needle rigid length – 17 mm
Total needle length (excl. handle) – 1 200 mm
Tag diameter – 1.1 mm
Tag length – 10 mm
Surgical suture type – EP1.5 (USP 4-0) braided polyester (non absorbable)
Maximum duration of use – 4 hours

Intended use

TAPES is a suturing device using a flexible endoscope. For use in natural orifices.

Contraindications

Any contraindication to endoscopy; gastric ulcer; perforations; severe gastritis or any pre-existing conditions that may lead to fragile mucosa; bleeding; oesophagitis; colitis; hiatal hernia; women who are pregnant, trying to become pregnant or nursing; children; elderly; addiction (drug or alcohol); bleeding disorders, rectitis, varices; portal hypertension in the stomach; fibrotic sites; patients taking anticoagulation therapy

Warnings

In order to ensure the safe use of TAPES, it is essential to read this paragraph carefully.

- Before using TAPES, make sure that the package is properly sealed and includes all the components shown in the picture and that they are in perfect condition. If any component is missing or damaged, do not use TAPES.
- TAPES is a device for gastroenterology. It can only be used in natural orifices.
- TAPES can only be used by qualified medical staff (a gastroenterologist or paramedical staff under surveillance by a gastroenterologist) who has received the necessary training and certification to use it.
- Always keep at least one additional TAPES in stock before starting a procedure.
- This document describes the use of TAPES, not the clinical procedures that can be performed with it.
- TAPES is single-use and cannot be reused : reusing poses a threat of cross contamination between patients. Additionally, reusing device after releasing tags makes device useless.
- TAPES is single-use and cannot be sterilized : sterilization process used in hospitals has not been tested could damage the device.
- TAPES can only be used on patients under general anaesthesia.
- TAPES can only be used with adult triangulation platforms. See "Triangulation platform compatibility" for more details.
- If TAPES is placed outside a clean zone, it cannot be used.
- If one of the components of TAPES becomes detached and falls into a body cavity of the patient, retrieve it with endoscopic forceps if it is judged dangerous for the patient's health.
- If the triangulation platform does not allow large enough plications to be formed, the suture may not hold.

Precautions

- Never make sudden movements, especially during insertion and removal. This could hurt the patient.
- The needle must not be apparent during the introduction and removal of the triangulation platform. This could hurt the patient.
- TAPES must always be manipulated under visual surveillance when the needle is outside the triangulation platform, in the cavity to be treated. Not respecting this precaution could lead to dangerous perforations of neighboring organs.
- Always wear appropriate protective clothing, as blood and other tissues carry an infection risk. Suitable protective clothing includes protective goggles, a mask and waterproof clothes and gloves.
- Do not use force to insert TAPES into the triangulation system. If there is resistance, remove it and investigate.
- When multiple tag chains are used, avoid sutures crossing each other.
- Do not insert TAPES into a bent channel. Always introduce TAPES into a channel that is as straight as possible.
- TAPES is a 1,9 mm device. Use it only in a suitable channel (up to 2,3 mm). To use it in a larger channel, use a catheter to avoid guiding issues.
- Always keep the handle firmly in the hand to avoid undesired movements.
- Always use CO₂ insufflation.
- Always have appropriate hemostasis tools available.

Adverse reactions

The following adverse reactions have been observed during endoscopic interventions involving needles and may happen after the use of TAPES:

- Perforation
- Pneumothorax due to CO₂ insufflation through iatrogenic perforation
- Nausea, vomiting, cramps
- Bleeding
- Minor peritonitis
- Minor bacteraemia

EN – Instructions for use

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from US Endoscopy and 6 mm monofilament oval snares from Endo-Flex GmbH. For other accessories, please test before use.

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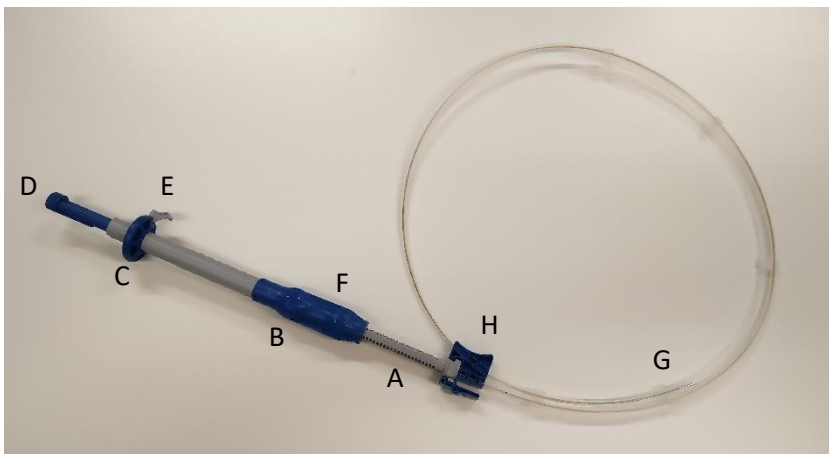
-  Product malfunction can be reported by e-mail to urgent@endotools.be. Please include your phone number in your e-mail.

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Technical information

Maximum diameter – 1.9 mm
Needle diameter – 1.75 mm
Needle rigid length – 17 mm
Total needle length (excl. handle) – 1 200 mm
Tag diameter – 1.1 mm
Tag length – 10 mm
Surgical suture type – EP1.5 (USP 4-0) braided polyester (non absorbable)
Maximum duration of use – 4 hours

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Contraindications

Any contraindication to endoscopy; gastric ulcer; perforations; severe gastritis or any pre-existing conditions that may lead to fragile mucosa; bleeding; oesophagitis; colitis; hiatal hernia; women who are pregnant, trying to become pregnant or nursing; children; elderly; addiction (drug or alcohol); bleeding disorders, rectitis; varices; portal hypertension in the stomach; fibrotic sites; patients taking anticoagulation therapy

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- Before using TAPES, make sure that the package is properly sealed and includes all the components shown in the picture and that they are in perfect condition. If any component is missing or damaged, do not use TAPES.
- TAPES is a device for gastroenterology. It can only be used in natural orifices.
- TAPES can only be used by qualified medical staff (a gastroenterologist or paramedical staff under surveillance by a gastroenterologist) who has received the necessary training and certification to use it.
- Always keep at least one additional TAPES in stock before starting a procedure.
- This document describes the use of TAPES, not the clinical procedures that can be performed with it.
- TAPES is single-use and cannot be reused : reusing poses a threat of cross contamination between patients. Additionally, reusing device after releasing tags makes device useless.
- TAPES is single-use and cannot be resterilized : sterilization process used in hospitals has not been tested could damage the device.
- TAPES can only be used on patients under general anaesthesia.
- TAPES can only be used with adult triangulation platforms. See “Triangulation platform compatibility” for more details.
- If TAPES is placed outside a clean zone, it cannot be used.
- If one of the components of TAPES becomes detached and falls into a body cavity of the patient, retrieve it with endoscopic forceps if it is judged dangerous for the patient’s health.
- If the triangulation platform does not allow large enough plications to be formed, the suture may not hold.

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