[D-TAPES-MDR-IFU--v4 – July 2024]

EN – Instructions for use p.1

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SR – Uputstvo za upotrebu p.4

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## EN – ENGLISH – Instructions for use

### Endoscopic tissue approximation device

*Do not use if package is damaged or unintentionally opened.*

### Terminology and package contents

- A. Fixing handle
- B. Needle pusher
- C. Distal tag pusher
- D. Proximal tag pusher
- E. Safety pin
- F. Brake screw
- G. Needle and tag chain (in the dispenser)
- H. Dispenser attachment

### Technical information

Maximum diameter – 1.9 mm

Needle diameter – 1.75 mm

Needle rigid length – 17 mm

Total needle length (excl. handle) – 1 200 mm

Tag diameter – 1.1 mm

Tag length – 10 mm

Tag material - PEEK

Surgical suture type – EP1.5 (USP 4-0) braided polyester (non absorbable)

Maximum duration of use – 4 hours

### Intended use

TAPES is a long-term, single use suturing medical device for endoscopic gastroplasty using a flexible endoscope and a triangulation platform (endomina® v2).

### Intended users

The intended users are:

- Nurse(s) trained on the device
- Nurses in training accompanied by clinical field specialist
- Gastro-enterologists trained on the device
- Gastro-enterologists in training accompanied by experienced gastro-enterologist(s)

All users must understand at least one of the languages in the IFU.

### Target patient population

Adults (18 and 65 years)

### Clinical indication

Obesity (restrictive surgery on the stomach)

### Clinical benefits of TAPES used in combination with endomina® v2

▪ Mean excess weight loss (EWL) ≥ 25% at 12 months.

▪ Improved satiety: >30% volume intake reduction

### Performance characteristics of the device

TAPES offers full-thickness tissue approximation made of double plications.

TAPES is easy to use less than 25 procedures needed before operator is able to decrease procedure time.

### Contraindications

Any contraindication to endoscopy: gastric ulcer; perforations; severe gastritis or any pre-existing conditions that may lead to fragile mucosa; bleeding; oesophagitis; hiatal hernia; women who are pregnant, trying to become pregnant or nursing; children; elderly; addiction (drug or alcohol); bleeding disorders; varices; portal hypertension in the stomach; fibrotic sites; patients taking anti-coagulation therapy.

### Warnings

In order to ensure the safe use of TAPES, it is essential to read this paragraph carefully.

▪ Before using TAPES, make sure that the package is properly sealed and includes all the components shown in the picture and that they are in perfect condition. If any component is missing or damaged, do not use TAPES.

▪ TAPES is a device for endoscopic gastroplasty. It can only be used through the mouth.

▪ TAPES can only be used by qualified medical staff (a gastroenterologist or paramedical staff under surveillance by a gastroenterologist) who has received the necessary training and certification to use it.

▪ Do not touch needle tip during TAPES removal and disposal : it may be contaminated with potentially infectious substances from the patient.

▪ Always keep at least one additional TAPES in stock before starting a procedure.

▪ This document describes the use of TAPES, not the clinical procedures that can be performed with it.

▪ TAPES is single-use and cannot be reused : reusing poses a threat of cross contamination between patients. Additionally, reusing device after releasing tags makes device useless.

▪ TAPES is single-use and cannot be resterilized : sterilization process used in hospitals has not been tested and could damage the device.

▪ Do not use TAPES after expiration date, it could lead to device malfunction (i.e. needle detachment).

▪ If TAPES has been exposed to environmental conditions outside of those specified on the label, do not use TAPES.

▪ If TAPES is placed outside a clean zone, it cannot be used.

▪ TAPES can only be used on patients under general anaesthesia.

▪ TAPES can only be used with endomina® v2 platform. See "Triangulation platform compatibility" for more details.

▪ If one component of TAPES becomes detached and falls into a body cavity of the patient, retrieve it with endoscopic forceps if it is judged dangerous for the patient's health.

Note 1: except the needle, components are not radio-opaque.

Note 2: the needle detaching from the device due to a malfunction might lead to perforation of a vital organ or non-vital organ including esophagus and/or stomach.

▪ If the triangulation platform does not allow large enough plications to be formed, the suture may not hold.

### Precautions

▪ Patient should be placed in supine position to avoid entrapment of neighboring organs

▪ Never make sudden movements, especially during insertion and removal. This could hurt the patient.

▪ The needle must not be apparent during the introduction and removal of the triangulation platform. This could hurt the patient.

▪ TAPES must always be manipulated under visual surveillance when the needle is outside the triangulation platform. Not respecting this precaution could lead to dangerous perforations of neighboring organs.

▪ Always wear appropriate protective clothing, as blood and other tissues carry an infection risk. Suitable protective clothing includes protective goggles, a mask and waterproof clothes and gloves.

▪ Do not use force to insert TAPES into the triangulation system. If there is resistance, remove it and investigate.

▪ When multiple tag chains are used, avoid sutures crossing each other.

▪ Do not insert TAPES into a bent channel. Always introduce TAPES into a channel that is as straight as possible.

- TAPES is a 1.9 mm device. Use it only in a suitable channel (up to 2,3 mm). To use it in a larger channel, use a catheter to avoid guiding issues.
- Always keep the handle firmly in the hand to avoid undesired movements.
- Check the needle tip integrity after removal and ensure that the tip is still attached to TAPES device. If needle tip is missing retrieve it from the patient using sharp foreign body extraction method.
- Always use CO<sub>2</sub> insufflation.
- Always have appropriate hemostasis tools available.
- A maximum of 15 TAPES can be used per patient and per procedure.

### Potential harm for the patient

The following harms may occur during or after the use of TAPES:

- Acute inflammatory tissue reaction
- Bleeding requiring hemostasis management during or after the procedure
- Bleeding (stomach, liver, bowel, gallbladder, pancreas, spleen, omentum)
- Cross contamination
- Delayed intervention
- Infection
- Irritation
- Perforation (esophageal and/or stomach)
- Perforation (pharynx, bowel, bladder, spleen, uterus)
- Perforation (heart, lungs, pancreas, liver, aorta)
- Pneumoperitoneum
- Re-intervention might be required
- Sepsis
- Small mucosal tear
- Strong abdominal pain
- Third party contamination
- User injury

### Side effects

Nausea, vomiting, cramps

### Procedure

The procedure described here is a check-list for an operation by an experienced user.

1. Check that the TAPES package is not damaged and is perfectly sealed. If any anomaly is detected or if the packaging has been unintentionally opened, do not use TAPES; send it back to Endo Tools Therapeutics.
2. Insert endomina® v2 at the site to be treated.
3. Unpack TAPES and check visually that it is complete and in good condition. Check that the tags are in place and the handle has not been actuated. Remove TAPES from the dispenser by turning the fixing handle. Always hold the needle and the tubing and keep it pointed away from yourself and other people. Do not uncover or unwrap the needle until it is time to use TAPES.

4. Check that the needle pusher slides in the fixing handle and check the brake screw. The brake screw can be tightened at any time to lock needle pusher movement.
5. Gently introduce TAPES into endomina® v2, using lubricating gel if necessary. Do not use force. Check that the angling of endomina® v2 allows insertion.
6. Fix TAPES to endomina® v2 with the connector.
7. Actuate endomina® v2 to position the TAPES needle correctly.
8. Check visually that the TAPES needle slides correctly out of endomina® v2.
9. Grab and pull tissue with the appropriate grasping device.
10. Pierce the first tissue by pushing the handle.
11. Tighten the brake screw before releasing the tags
12. Remove the safety pin and release the distal tag using the distal tag pusher.

13. Pull back the distal tag pusher flush with the proximal tag pusher.
14. Pull the needle back into endomina® v2 by pulling the handle.
15. Move endomina® v2 back to loosen the suture if necessary.
16. Grab and pierce the second tissue.
17. Release the proximal tag using the proximal tag pusher. Check that the entire suture has been released. Make back and forth movements with the tag pusher if necessary.
18. Pull back the proximal tag pusher to its rest position.
19. Using the handle, pull back the needle into endomina® v2.
20. Straighten endomina® v2 to allow needle removal. Always keep visual control.
21. Take TAPES out of endomina® v2. Do not use force.
22. With the help of a polypectomy snare, tighten the tag chain by pulling on the suture tightening ball. Tighten until the tags exert suitable pressure on the tissues.
23. With a cutting tool, cut the free end of the suture. Make sure you do not cut the suture between tags.

### End of procedure

1. Check that all the components are present (excluding tags and suture).
2. Dispose of TAPES in the sharp hospital waste bin. Make sure the disposal container is made for disposing of sharp objects and make sure there is enough room in the container for your object to fit. Replace containers when they are two-thirds full or according to hospital rules. If a needle is sticking out of the container, do not push it in with your hands. Call to have the container removed.

### Emergency procedure

If TAPES gets stuck in endomina® v2 or tissue, use the following procedure:

- If TAPES is stuck in tissue:
  1. Release the proximal tag to release the needle from the tissues.
  2. If necessary, cut the tag or suture with an endoscopic guillotine or scissors to release the needle.
  3. If necessary, retrieve the tags and suture with endoscopic forceps.
- If TAPES is stuck in endomina® v2:
  1. Completely remove the distal and proximal tag handles.
  2. Check that the angle of endomina® v2 allows TAPES to slide.
  3. If TAPES is still stuck, the needle does not slide out of the operating channel and endomina® v2 allows it, remove everything together (endomina® v2 and TAPES) and investigate the reasons for the blockage.
  4. If the needle is out of the channel or endomina® v2 does not allow for combined removal, cut the needle from its tube e.g. with argon plasma.
  5. Retrieve the needle with the endoscope and foreign body forceps.
  6. Remove TAPES from the operating channel.

If TAPES is damaged during the operation and a part stays in the stomach, take it out of the patient.

### Tag chain removal

In case of necessity, tag chain suture can be cut with a cutting tool between the tags. Tag chain elements should then be removed with a grasper. Dispose of the tag chain in an appropriate trash (for hospital waste)

### Storage

TAPES must be stored in its sterile package, in a clean and dry place, away from sun exposure and at ambient temperature.

The sterile package should not be stored underneath other packages that could squash it.

### Repairs and modifications

TAPES has no parts that can be repaired by the user. TAPES must not be dissembled, modified or repaired by the user. This could hurt patients or damage other medical devices.

### Triangulation platform compatibility

TAPES has been validated with endomina® v2 triangulation platform (endomina® v2)

There is no guarantee that instruments selected solely using maximum insertion portion width and working length will be compatible in combination.

### Other endoscopic tools required for the procedure

TAPES can be used with endoscopic tools through the channels of the endoscope.

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Any serious incident that has occurred in relation to the medical device should be reported by e-mail to urgent@endotoools.be and to the competent authority of the country in which you are established  
Please include your phone number in your e-mail.  
Summary of Safety and Clinical Performance is available in the European database on medical devices : <https://ec.europa.eu/tools/eudamed>  
TAPES Basic UDI-DI: 542502928TDS1BZ  
IFU is available on [www.endotoools.be/ifu/](https://www.endotoools.be/ifu/)

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